

Wide Panel Screen - White

Item #081511104

Usage Instructions


Assembly:

- 1) Insert either the 2" casters or the crutch tips into the legs. Attach the legs to the main frame using the ¼" x 1½" bolts and lock nuts. Tighten nuts. Please note that the bolts must fit securely into holes to assure that the leg mount will be steady. Use a Phillips Screwdriver to do so. Also, make sure that the legs are installed on the outside of the main frame.
- 2) Confirm that the screen is level. If not, bend the legs slightly to make sure that they are all flat on the floor.

Note: Check all fasteners for tightness periodically.

To replace the vinyl panel, grip the curtain rod at the end and slide the curtain rod towards you. This will free the curtain rod from the frame.

For cleaning, use a mild liquid soap and water.

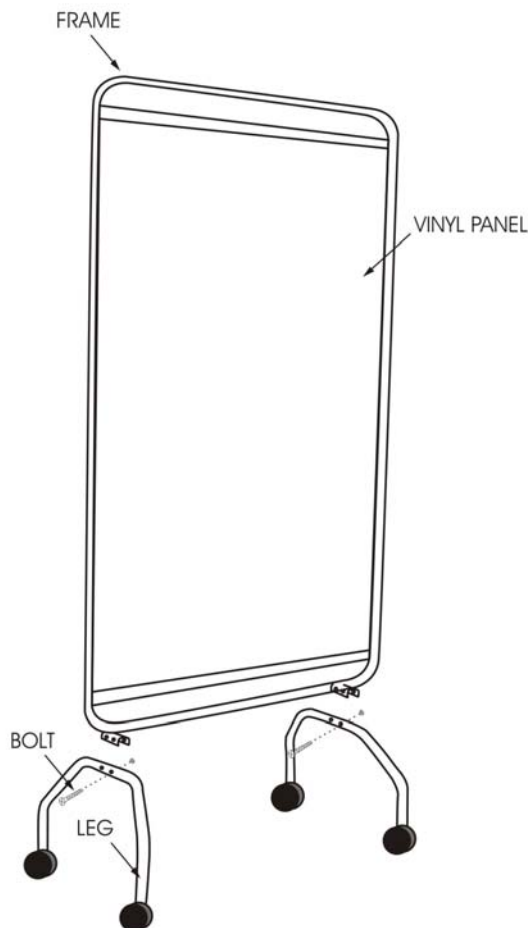

Sammons Preston

A Patterson Medical Company
1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310

WARRANTY

This product is guaranteed to be free from defects in material and/or workmanship under normal use and service for a period of one year from date of purchase. The above guarantee is null and void if unauthorized repair and/or modification is performed on the unit.