


Retractable Monofilament

The 5.07 monofilament is primarily indicated for testing the feet of diabetic patients for loss of protective sensation associated with peripheral neuropathy. Diabetic patients who are unable to feel the pressure of the applied 5.07 filament should be considered at risk of developing pressure or neuropathic ulcers and should be referred to a specialist for further assessment. Do not use on ulcerated or necrotic tissue. Clean with an alcohol swab between patients. Do not autoclave. Store at room temperature.

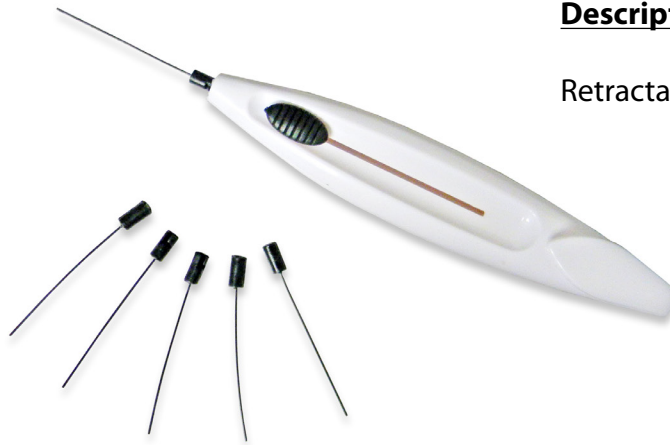

Description


Patterson

Retractable Monofilament

081599505

Disposable Monofilaments

Use a smooth motion to touch the filament to the skin on your foot. Touch the filament along the side of and NOT directly on an ulcer, callous, or scar. Touch the filament to your skin for 1-2 seconds. Push hard enough to make the filament bend.

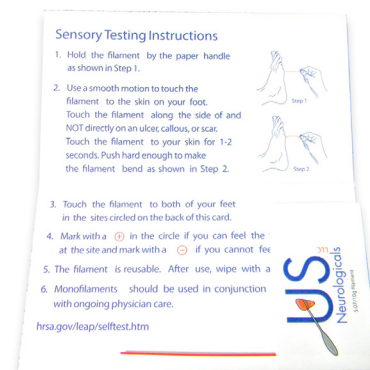

Description

Patterson

Disposable Monofilament (10pk)

081599489

Disposable Monofilament (100pk)

081599497

Patterson Medical
1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440
1-800-323-5547
www.pattersonmedical.com