## Rolyan® Anti-Spasticity Ball Splint:

A673-1 through A673-2

### Ball:

A373-4

### Tool for pulling material between fingers:

A437-1

#### INDICATIONS

- To help prevent deformity from mild to moderate hypertonicity
- May be used following CVA and traumatic brain injury, and with Parkinson's disease

#### CONTRAINDICATIONS

- Not for use with severe edema
- Not for use with wrist or hand fractures
- Not for use over open wounds and infected areas

#### PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

#### CARE AND CLEANING


- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint may be cleaned with soap and lukewarm water. The straps may be washed with soap and water, but the adhesive portion should not be moistened. The splint and straps should be allowed to dry thoroughly before reapplication of the splint.

#### PROCEDURE


The splint blank can be applied to either hand—simply invert the splint blank.

**Fabricating a volar splint** (see instructions 11–22 for a dorsal splint)

1. Heat splint blank in water heated to approximately 160°F (70°C) until pliable. (Refer to the instructions for Rolyan splinting materials for information about working with specific splinting materials.)
2. Remove from heat pan and place on a towel on a flat surface. Blot excess water.
3. Note that the wider portion of the splint is the radial side.
4. To assist with contouring the web spaces, flare the entire pan portion of the splint.
5. Put splint blank onto a ball covered with a wet paper towel. Stabilize the ball with a stationary object or exercise putty.
6. Position the patient's hand symmetrically onto the splint blank (A).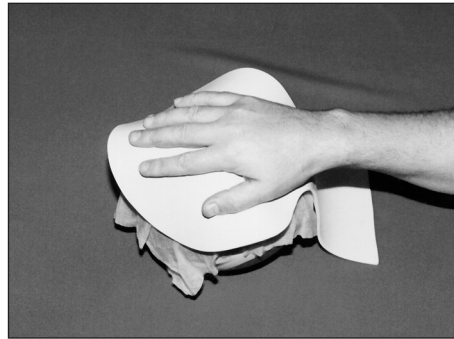

A

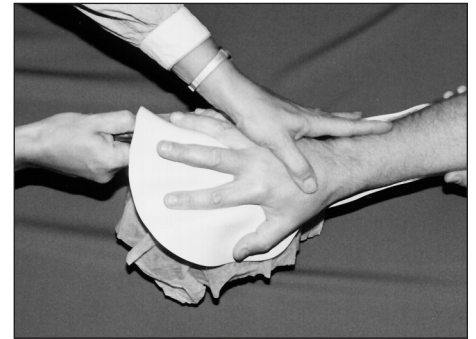B

C

7. Mold the splint to the forearm. If desired, wrap the forearm lightly with an elastic bandage to hold the splint in place.
8. Using the tool for pulling material between the fingers (Product No. A437-1), a dowel, or another wand-type device, pull the material aggressively through the finger web spaces (B).
9. The splint can be trimmed and the edges flared if necessary.
10. Apply appropriate straps. A completed volarly applied splint is shown at the top in C.

#### Fabricating a dorsal splint

11. Note that the wider portion of the splint is the radial side. Place hand on splint blank and mark the width of the hand at the MCPs and the thumb at the MCP.
12. Heat splint blank in water heated to approximately 160°F (70°C) until pliable. (Refer to the instructions for Rolyan splinting materials for information about working with specific splinting materials.)
13. Remove from heat pan and place on a towel on a flat surface. Blot excess water.
14. To assist with contouring the web spaces, flare the entire pan portion of the splint.
15. Cut slits as marked.

16. Roll back splinting material at the slits so the folds will be to the outside of the completed splint (the proximal fold should face up and the distal fold down). Roll back just enough material so the digits fit through the slits.
17. Position the patient's hand symmetrically in the splint blank and put onto a ball covered with a wet paper towel. Stabilize the ball with a stationary object or exercise putty.
18. Mold the splint to the forearm. If desired, wrap the forearm lightly with an elastic bandage to hold the splint in place.
19. Using the tool for pulling material between the fingers (Product No. A437-1), a dowel, or another wand-type device, pull the material aggressively through the finger web spaces (B).
20. Roll back the dorsum of the splint to clear the MCPs.
21. The splint can be trimmed and the edges flared if necessary.
22. Apply appropriate straps. A completed dorsally applied splint is shown at the bottom in C.

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**Patterson Medical, Ltd.**