## Rolyan® Aquatubes® Splinting Tubes:

A962-51, A962-52, A9652-53 and 5560-41

Rolyan Aquatubes Splinting Tubes are for use with splinting materials as line guides, outriggers, and hinges in dynamic splints. They can also be used to form figure-8 splints.

### PRECAUTIONS

- **After heating splinting material, always check its temperature before applying it to the patient.**

### CARE AND CLEANING

- The material will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.

- The material can be cleaned with soap and lukewarm water. Allow it to dry thoroughly before reapplication to the patient.

### PROCEDURE

1. If splint base is coated to prevent accidental bonding, remove coating by scraping the surface or brushing it with an acetone material.
2. Soften in a heat pan of water at a temperature over 140°F (60°C).
3. Form by hand.
4. With a heat gun, spot heat the tube ends and the splint where they will be bonded until tacky. Make sure all areas to be bonded are hot and dry.

5. Mount to splint base with firm finger pressure.
6. Aquatubes can be revised to accommodate patient progress by reheating and remolding them.
7. To make a hinge, attach the softened Aquatubes to the proximal and distal bases of the splint. While the tube is still soft, pinch with needle-nosed pliers at the desired axis of the hinge. Hold the pliers in place until the material has cooled.

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**Patterson Medical Ltd.**