



Washable, reusable fabric face mask with Silvadur™ antibacterial protection to guard face, nose and mouth against respiratory droplets and splashes.

- Provides a physical barrier protection against larger respiratory droplets, germ particles, and splashes for the user and the environment. While this mask will provide some protection against, it will not act as a barrier against smaller respiratory droplets or germ particles.
- Silvadur™ antibacterial protection, with the elemental power of silver, is integrated into the fabric.
- Washable multiple times without losing effectiveness, even at high temperatures - $\geq 160^{\circ}\text{F}$ ($\geq 71^{\circ}\text{C}$).
- May be steam cleaned at temperatures between 212°F (100°C) and 270°F (132°C).
- Ironing is permitted at temperatures up to 365°F (185°C).
- Soft, skin-friendly TriFlex™ fabric with 3D Stretch for optimal comfort.
- Lightweight, one-piece design covers the nose and mouth, with comfortable ear straps to ensure the mask stays in place.
- Good airflow and breathability.
- Can be worn on its own or over a respirator
- One size fits all

Effective Antibacterial Protection Integrated Into Every Mask

SILVADUR™ unites silver's elemental powers with a patented delivery system to create an extremely effective, highly durable bio-activated antibacterial agent, which delivers silver ions to the fabric surface.

Silvadur™ Is Non-Toxic

SILVADUR™ is approved as non-toxic by the international Oeko-Tex Association. SILVADUR™ is registered with the US EPA (EPA Reg. No. 707-313.) and its ingredients is notified and supported under the EU BPD.

Reusable, And Washable Multiple Times, Even At High Temperatures

SILVADUR™ stays on the fabric and remains active for multiple washings. SILVADUR™ won't detach or degrade at high temperatures, and won't discolor fabrics.

Soft, Skin-Friendly TriFlex™ Fabric

Made of our patented post-surgical fabric for a comfortable wearing experience when used on its own or over N95 masks and respirators.

*This product is non-sterile and has not been tested to N95 standard. It is not a medical device. It is not intended to be used as a respirator. Each mask is intended for use by one person only and must not be shared. This mask has not been tested for effectiveness against the COVID-19 virus.



Proudly made in the USA by The Marena Group, LLC, a medical device manufacturer. Marena is ISO: 9001, ISO 13485:2016 and GMP certified.

Marena Antibacterial Face Mask

Styles: FMRAB-10PK | FMRAB-50PK

Colors:  BEIGE  PINK

Sizes: Universal - One size fits all

AVAILABLE EARLY APRIL 2020



Instructions For Use

When putting mask on, place shiny side to face, with pleats facing downward. Put loops over ears, and adjust to tightly cover the nose and mouth (or respirator if combined). If a tighter fit is required, the loops can be twisted before placing over ears. To remove contaminated mask, avoid touching outer layer of mask or your face. Remove from ears, only touching loops, and immediately put into washer, sterilizer or safe storage. Clean mask by washing or sterilizing before re-use every time. Sanitize hands immediately after handling contaminated masks each time. Clean after multiple use or if soiled. Discard if damaged or torn.

Multiple Pack Sizes Available

Mask are available in the following quantities:

10-PACKS

ITEM NUMBER	DESCRIPTION
FMRAB-10PK-U-H	Reusable Antibacterial Face Mask - 10 Pack - Universal – Beige
FMRAB-10PK-U-LP	Reusable Antibacterial Face Mask - 10 Pack - Universal – Pink

50-PACKS

ITEM NUMBER	DESCRIPTION
FMRAB-50PK-U-H	Reusable Antibacterial Face Mask - 50 Pack - Universal – Beige
FMRAB-50PK-U-LP	Reusable Antibacterial Face Mask - 50 Pack - Universal – Pink

Packaging Info

Dimensions: 24.1 cm x 27.9 cm (9 1/2" x 11")

Weight: 0.115 kg (0.5 lbs)

Used for 10-piece and 50-piece configurations



For more information, contact us at info@marenagroup.com or call 1-888-462-7362