



Examine Real Quality™

FOLDING MAT PLATFORM

MODEL 243




ASSEMBLY INSTRUCTIONS

ASSEMBLY INSTRUCTIONS FOR FOLDING MAT PLATFORM

MODEL 243

MAXIMUM WEIGHT CAPACITY 500 LBS

Thank you for counting on Clinton to meet your medical equipment needs by purchasing this quality equipment. All Clinton equipment is designed and manufactured to provide years of dependable service because we want you as a customer now and in the future. Below are easy to follow instructions on how to properly assemble your equipment. Please contact us at 800-441-9131 if you experience any difficulty with the assembly or have questions.

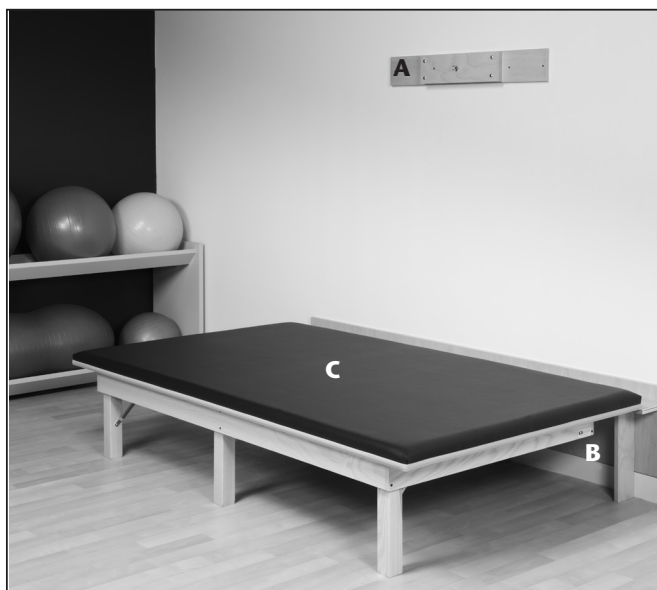

Contents

- A. Chain Board
- B. Back Board
- C. Mat Platform

Hardware

- (2) 2 1/2" Phillips Head Bolts
- (2) Locknuts
- (4) 3/4" Phillips Head Machine Screws
- (2) 2 1/4" Hex Head Bolts
- (1) 2" Hex Head Bolt
- (12) Washers
- (1) Acorn Nut
- (4) Black Star Knobs
- (1) Tension Knob

Tools Required

- Phillips Screwdriver

Time Required

- 20-25 Minutes

1

Place the Back Wall into preferred location and have professionally installed using appropriate hardware in accordance with wall type. Lower legs on Platform and lock into down position in preparation for the next step. Note- the Back Wall must sit on the floor and be flush against the wall.



2

Make sure chain is connected between the Chain Board and the Mat Platform. Place the Platform in the stored position and place Chain Board on the wall at a height where the chain is tight. This will be the appropriate height for the Chain Board to be professionally installed. Use appropriate hardware in accordance with wall type.



3

Place Platform on installed Back Wall and align holes in hinge with those in Back Wall. Install and tighten hardware.



4

Usage Position- Lower legs and lock into down position, unhook chain and carefully lower until the legs rest on the floor. Never allow the Platform to drop to the floor from any height.



5

Clean with disinfectant according to Cleaning Guidelines on back page prior to use.

Intended Use

IMPORTANT: Clinton Wall Mounted Mat Platform, model 243, is specifically designed and engineered for physical therapy use only. Persons using the mat platform should be observed by trained medical/physical therapy personnel. Never leave anyone on the mat platform unattended. Combination of weight and force exerted on the mat platform during therapy is not to exceed a total of 500 lbs. Using this equipment for unintended purposes could result in damage to the equipment and serious injury.

Warranty

Clinton Industries warrants manufactured products to be free of defects in materials, workmanship or design under normal use for five years on all products, from the date of purchase. This warranty does not include damage resulting from accident, abuse or misuse, alteration, or negligence of the product and expressly excludes normal wear of parts (including but not limited to vinyl). Contact Clinton for full warranty details.

Care & Cleaning

Clinton products are built to be durable. With proper care, your Clinton equipment will provide years of superior looks and performance. Clean platform with a soft cloth and any mild cleaning agent that may be required for sanitation. NEVER use abrasive scrubbing agents that may scratch the surfaces. **Prior to cleaning entire surface, test cleaning agents on an inconspicuous area. Note that certain cleaning agents can be harmful to the surfaces of the product.** Periodically check all fasteners to make sure they remain secure.



Examine Real Quality™

1140 East Edison St., York, PA 17403

800-441-9131 • 717-848-3519

Toll-Free Fax 866-522-1233

Fax 717-843-5871

www.clinton-ind.com