User instructions

[®]
UCAST

Contents

AR	تعليمات المستخدم	4
CS	Uživatelské pokyny	8
DA	Brugervejledning	12
DE	Benutzeranleitung	16
EN	User Instructions	20
ES	Guía de usuario	24
ET	Käsiraamat	28
FI	Käyttöohje	32
FR	Mode d'emploi	36
HE	הוראות למשתמש	40
HI	उपयोगकर्ता गाइड	44
HR	Felhasználói utasításokat	48
IT	Guida utente	52
JA	マニュアル	56
KO	사용 설명서	60
NL	Gebruiksaanwijzing	64
NO	Brukerhåndboken	68
PT	Guia de usuario	72
RU	Руководство пользователя	76
SV	Bruksanvisning	80
ZH	使用说明	84

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EN • UCAST® is a novel, two-part supporting device, which can be personalized by reshaping the rigid splint component.

UCAST products are comprised of two components:

1. Heat-moldable, rigid, and non-toxic splint(s)
2. Self-attaching support fabric

PURPOSE OF USE

UCAST is intended and suitable for immobilization of limbs or head and neck area. UCAST is robust and can be used in casting, splinting, or as customized orthosis in active rehabilitation

PROPERTIES

UCAST has two components: 3D stretching fabric with self-attachment properties, and a rigid skeleton of moldable and re-moldable biomaterial. The UCAST material is as hard and rigid at room or body temperature, but becomes freely moldable, pliable, and adhesive when heated to 65°C/150°F. UCAST material achieves its mechanical strength when cooling down to body temperature. Hardening can be accelerated by external cooling if necessary. Once set, the UCAST splint is lightweight, durable and almost totally X-ray transparent. UCAST products are non-toxic, and can be handled with no specific protective gloves. They are pre-shaped for any given indication, and the splint material has vents for breathability. All the fabrics are resistant to odors and water.

INSTRUCTIONS FOR USE

1. Heat the UCAST splint component(s) without fabrics in the UCAST heating device, according to the provided heating suggestions.
2. UCAST splints are ready to be used once they are completely soft.
3. Place the fabric on an even surface and place the warmed splint on it as shown in the UCAST packaging. This final splint with fabrics and UCAST skeleton can still be molded freely in 3D.
4. Ensure that the splint is not too hot, especially if heated in hot water (described later). Remember that heat burden is higher in products with a 4 mm splint skeleton. Test the splint on your own skin first. If necessary, wait a moment for the material to cool down.
5. Place the product on the patient's skin, splint away from the skin. Mold the UCAST splint into the desired shape. If necessary, you can assist stabilization of the cast/splint by wrapping the support fabric gently before tightening the application. Do not apply a thick layer of bandaging before the material has cooled down - this may prevent the heat from dissipating away from the skin.
6. Extreme care must be taken when treating patients under anesthesia, patients with neuropathic problems, or any other patients with impaired skin sensation.
7. Children's extremities are more sensitive to heat burden, especially in circular or almost circular applications.
8. Finish off the cooled and solidified application by wrapping the support fabric around the application. Make sure to tighten this well. To make it circumferential, add the 2-sided glue tapes in the end, this way the application can't be removed.
9. If necessary, you can reshape the splint skeleton by reheating it with the UCAST Express heater. **Important: BEFORE reheating, ALWAYS REMOVE the support fabric and any additional padding materials from the UCAST skeleton.**

UCAST SPLINT HEATING TIMES

Flatbed Heater: 5–10 min
Express Heater: approx. 100 seconds
UCAST Knee remolding:
15–20 min in Flatbed Heater,
3 min in Express Heater

IMPORTANT:

DO NOT heat the support fabric in the Express Heater, to avoid fire hazards.

UCAST SPLINT MOLDING AND SETTING TIMES

Time moldable: 3–5 min
Fully rigid: 5–10 min
UCAST Knee remolding: moldable 5 min, 10–15 min until fully rigid

HEATING UCAST WITH HOT WATER

UCAST can also be heated in a water bath. Place the whole package in hot water. Be careful not to burn yourself. Make sure the package isn't opened or damaged before submerging it in water. Follow the instructions on the package.

WATER TEMPERATURES AND INDICATIVE HEATING TIMES

65°C/149°F: 2 min 15 seconds
80°C/176°F: 1 min 50 seconds
100°C/212°F: 1 min 15 seconds

Check that the splint is soft and formable in the bag before opening the sealed bag. If reheating the splint in a water bath, make sure that the bag is fully sealed and waterproof.

CLEANING INSTRUCTIONS

The product may be hand-washed in 86°F/30°C water with mild detergent and air-dried. Do not put in the dryer or use any other heat source to dry. Rinse thoroughly to get rid of detergent.

REMOVING THE PRODUCT, STORAGE AND DISPOSAL

- UCAST splints are removed by removing any possible bandage and opening the self-adhesive fabric structure. In certain products, the Woodcast skeleton itself has to be opened. This can be done with normal cast scissors. In thick skeletons, an oscillating saw can be used with precaution.
- Store UCAST products in a dry place, protected from humidity, at a temperature lower than +40 °C and out of direct sunlight. Used UCAST splints can be recycled as energy waste.
- UCAST is self-adhesive only when heated. When combining various products, make sure that no padding or bandaging materials are left between the products, and that you press the products together with sufficient force to ensure proper adhesion.
- Always remove the splint from the patient for the entire time that you reheat the splint. When reheating the materials, you must also ensure that any padding, including the support fabric, is heat-proof or taken away.
- **Important: BEFORE reheating, ALWAYS REMOVE the support fabric and any additional padding materials from the Woodcast skeleton.**
- Make sure that the UCAST application you are using is sufficiently strong for the indication in question.
- Severe adverse events must always be reported to the product manufacturer and the national supervisory authority.

PATENT PENDING

WARNINGS AND PRECAUTIONS

- Always ensure that the UCAST splint is not too hot for the patient!
- Beware when treating patients with impaired sensation, or patients under anesthesia.
- Risk of excessive temperature load is increased in children's limbs, and when using 4 mm splint material (e.g., knee UCAST)
- Do not apply the splint directly onto the skin. Always use at least the support fabric that comes with the product. It is highly recommended to use an additional temporary stocking when molding the warmed splint to fit a patient's individual anatomy.
- Do not heat UCAST products in a water bath without the original, intact, and sealed package.

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© 2022 Dassiet Oy