

Carex® Ergo Offset Cane Instructions

Indications for Use

This cane is intended to assist individuals with limited mobility by providing support, balance, and stability during walking. It is suitable for users who experience:

- Minor to moderate balance impairments
- Fatigue or weakness in one or both lower limbs
- Post-operative recovery requiring additional support
- Conditions such as arthritis, injury recovery, or neuromuscular disorders

The cane is designed for use on flat and stable walking surfaces. It should only be used by individuals who can bear partial weight on at least one leg and have sufficient upper body strength to manage and control the cane safely.

Cleaning and Maintenance Instructions

Cleaning:

- Clean the cane regularly using a soft cloth dampened with mild soap and water.
- Avoid using harsh chemicals, bleach, or abrasive cleaners, as they may damage the surface or materials.
- Dry the cane thoroughly with a clean towel after cleaning to prevent moisture buildup or corrosion.

Maintenance:

- Inspect the cane regularly for signs of wear or damage, including cracks, loose handles, or worn-out rubber tips.
- Ensure the rubber tip is securely attached and not worn down; replace it immediately if it shows signs of excessive wear or decreased traction.
- Check any height adjustment mechanisms to ensure they are securely locked and functioning properly.
- Store the cane in a dry, cool environment when not in use to prevent rust or warping.

For continued safe use, do not attempt to modify or repair the cane beyond the manufacturer's guidelines. Contact the supplier or manufacturer for replacement parts or further assistance.