

Material

Rolyan® Ezeform®

Temperature

$\frac{1}{16}$ " (1.6mm) 150° to 160° F (65° to 70°C) water for 35 seconds

$\frac{3}{32}$ " (2.4mm) 150° to 160°F (65° to 70°C) water for 30 to 45 seconds

$\frac{1}{8}$ " (3.2mm) 160° to 170° F (70° to 75°C) water for 1 minute



Stretch

Maximum resistance

Time

$\frac{1}{16}$ " (1.6mm) 1 to 2 minutes of working time

$\frac{3}{32}$ " (2.4mm) 2 to 3 minutes of working time

$\frac{1}{8}$ " (3.2mm) 4 to 6 minutes of working time

**LATEX
FREE**